


HURRICAINE®

Luer-Lock Dispensing Cap

Luer-Lock Dispensing Cap provides neat and easy transfer of HurriCaine® Topical Anesthetic Liquid to any standard-size luer-lock syringe.

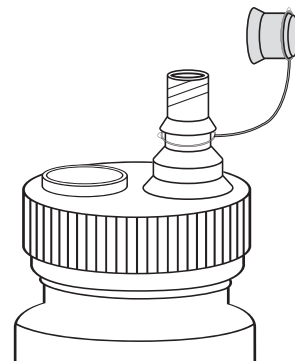


HurriCaine Topical Anesthetic Liquid

EASY TO ASSEMBLE!


- Resealable threaded cap prevents spills
- Fits all standard-size luer-lock syringes

STEP 1

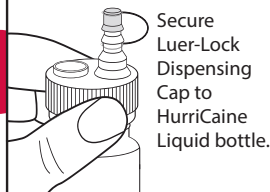


Secure Luer-Lock Dispensing Cap to HurriCaine Liquid bottle.

STEP 2

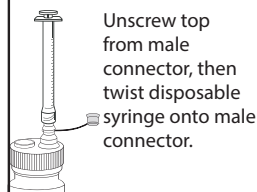

Unscrew top from male connector, then twist disposable syringe onto male connector.

STEP 3

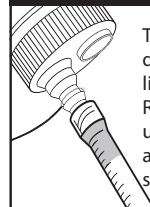

Turn bottle upside down and draw liquid into syringe. Return bottle to upright position and unscrew syringe from bottle.

STEP 4



Store in upright position only.

ORDERING INFORMATION:

Product	Item/NDC #	Description
HurriCaine Luer-Lock Dispensing Cap	0283-0609-43	Fits HurriCaine Topical Anesthetic Liquid bottles
HurriCaine Topical Anesthetic Liquid - Original Wild Cherry	0283-0569-31	1 fl. oz. bottle
HurriCaine Topical Anesthetic Liquid - Piña Colada	0283-1886-31	1 fl. oz. bottle
HurriPAK™ Irrigation Tips	0283-0908-21	Contains 20 tips
HurriPAK Refill Kit	0283-1107-07	Contains 10 syringes and 10 tips

Beutlich
PHARMACEUTICALS LLC
Since 1954

To place an order or for more information, please call us at 1-800-238-8542, M-F: 8:00 a.m. - 4:30 p.m. ET.
www.beutlich.com

HurriCaine is a registered trademark of Beutlich® Pharmaceuticals, LLC. HurriPAK is a trademark of Beutlich Pharmaceuticals, LLC. LCSS 524 0219